MidStream Pregnancy Rapid Test Instruction for Use (Self-Test)

INTENDED USE

MidStream Pregnancy Rapid Test is for rapid detection of Human Chorionic Hormone (hCG) in urine (see introduction). This is a self-test, meaning that it can be used by everybody, designed to determine pregnancy at early stages. It is qualitative test which means it does not give figures but only yes or no answers. Stage of pregnancy can not be determined by this test.

The sensitivity of this test is 25 mIU/ml, which will be reached 8-11 days after conception (see introduction).

INTRODUCTION

Conception usually happens in fallopian tube. The fetus, while is dividing, passes through the tube and will reach uterus. This will follow by implantation of fetus in uterus wall. Part of fetal cells will differentiate to become a special tissue which will eventually form placenta. This will secrete Human chorionic gonadotropin hormone (hCG).

Levels of hCG may be measured in the blood or urine. In the first 8 weeks of pregnancy, the hCG concentration in mother serum rises rapidly. Detectable amounts are present 8 to 11 days after conception, which is in the third week of pregnancy. This is counted from the last menstrual period. hCG reaches its highest level at about 8 to 10 weeks. After that hCG concentrations start to decline in serum and urine, and by the end of the 6 months it will reach its minimum level.

Generally speaking, the test can become positive in half of pregnant women on the first day of their missed period ^{1,2,3,4,5}.

PRINCIPLES OF EXAMINATION METHOD

MidStream Pregnancy Rapid Test is based on a method called "lateral flow immunochromatographic assay" (LICA). This method detects hCG in human urine. The pregnancy test contains reagents developed in mouse (Monoclonal anti hCG beta subunit antibody) and in the goat (polyclonal anti hCG alpha subunit antibody) which specifically react with this hormone. After putting urine in sample part of the test (see Test Procedure), the urine moves along the white paper like surface, called membrane. When hCG is present in the urine, it reaches the Test region of the membrane (identified by letter T), it will form a colored line. Absence of this colored line suggests a negative result. To serve as a procedure control, a colored line will always appear at the Control region (identified by letter C). Absence of this line indicates that the test has not been performed correctly.

PRECAUTIONS BEFORE USING THE TEST

- Read instruction for use carefully before using the test.
- Do not use the test beyond the expiration date printed on the box and on the pouch.
- Do not use the test if the pouch is torn, damaged or not properly sealed.
- The test device should remain sealed until use.
- Do not reuse the device. It just does not work.
- This test is for in vitro diagnostic use. **This means the test is for use only outside the body and it should not be taken internally.**
- This test has been designed for urine samples only. Please do not use other samples (blood, saliva,)
- Treat the urine specimen and use strip as if they are infectious.
- The reagent in this kit contains very small amount of sodium azide. However, if build up, it still can react with copper or lead in plumbing to form potentially explosive metal azide. **So do not discard them in the sink. Instead put the used test in plastic bag and discard in the dust bin.**

KIT CONTENTS

- Test device
- Silica Gel Pillow
- Instruction for use

URINE COLLECTION

Urine at any time of day may be used; however, the first morning urine generally contains the highest concentration of hormone. Reduce liquid intake about 2 hours before test to prevent dilution of urine.


TEST PROCEDURE

Please follow the test procedure step by step as described below. Please do not omit or add any steps or take shortcuts.

- Check expire date on the pouch and on the box. If expire date has passed, DO NOT USE THE TEST.
- Check the integrity of the pouch to see if the pouch is damaged, torn, or has not been properly sealed. If so, DO NOT USE THE TEST.
- Open the sealed pouch by tearing along the notch.
- Check the silica gel pack inside the pouch. It should contain colorless and blue crystals. No pink crystals should be present. If so, it means that at some point the pouch has been damaged. Also, if the Silica gel pack is missing from the package or has been torn, DO NOT USE THE TEST.
- If Pouch and the silica gel are good, then remove the device. If the cassette is damaged or its components are detached: DO NOT USE THE TEST. Please do not disassemble the device.
- If the device is intact, remove the cap and place the device on the level surface and let it to come to room temperature. Although the tests function properly at temperatures between 5o C to 30oC, yet normal room temperature is recommended.
- Use the test as soon as possible. Make sure the test is not out of the pouch for more than 60 minutes. Also do not disassemble the device.
- The window shape compartment is the part that should be exposed to urine sample and will be referred to as "sample part".
- For best result the MidStream sample will be required. This is the middle part of the urine. Start to urinate. At the middle of urination put the "sample part" of the test in urine stream without stopping the urination. Hold the test in place for 3-5 seconds. Then remove the device and put the cap back and place the device on leveled surface.
- Wait for at least 3 minutes then read the result.

READING OF EXAMINATION RESULTS

- The upper portion of the window next to letter C is called the Control region. Appearance of a pink-purple line here means that the test has worked properly.
- The lower portion of the window next to letter T is called the Test region. Appearance of a pink to purple line here means the test is positive.
- Negative Result: The appearance of only one pink-purple color line next to letter C means that most likely the pregnancy is not present. If in doubt the test should be repeated 2-3 days later.
- Positive Result: The appearance of two pink-purple color lines next to both C and T letters means that pregnancy is most likely to be present.
- Note: Once the lines appear, they will not change. That means negative or positive results will remain the same after 3 minutes.
- Invalid Result: If no pink-purple color line appears or if only appears next to letter T and not letter C, it means the test has not been performed properly. The result is invalid and the test needs to be repeated.
- Some causes of invalid results are: not following the directions correctly or the test may have deteriorated beyond the expiration date. Please repeat the procedure with new test device.

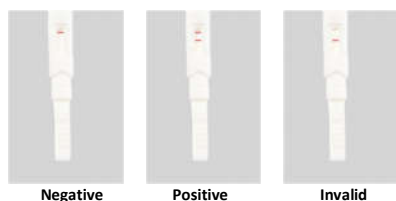NOTES TO BE REMEMBERED

- If a urine specimen is too dilute it may not contain enough levels of hCG.
- If the test is negative, in women strongly suspected to be pregnant, the test has to be repeated 2-3 days later, or perform a quantitative assay.
- The shade of red color in the test line region (T) will vary depending on the concentration of hCG present in urine. However, neither the quantitative value nor the rate of increase in hCG can be determined by this qualitative test.
- Some women may have borderline hCG levels (those showing a very faint line in the test line region). If these women repeat the test later, it may become positive, which indicated positive test. However, in some, the test will be negative. This can be attributed to falling hCG levels following a spontaneous or induced abortion. Natural termination of pregnancy occurs in 22% of clinically unrecognized pregnancies 5.
- A number of conditions other than pregnancy can cause elevated levels of hCG. For example, ovarian cysts or ectopic pregnancy (pregnancy outside of uterus). Some substances such as those used in antibody therapy treatments may give false test result. Also, very high concentration of hCG in later stages of pregnancies as well as some diseases may interfere with the test results. For these reasons, please remember the final and definitive clinical diagnosis of pregnancy should only be made by a physician after all clinical and laboratory findings have been evaluated.
- We kindly request that all users of this product report any defects or malfunction of the test by email (info@phoenixhealth.eu).
- As mentioned above this test will become positive in half of pregnant women on the first day of their missed period. Using the test sooner, may give false negative result, even though pregnancy is present

STORAGE AND STABILITY

- The test device can be stores at 5-30o C in the intact sealed pouches.
- Keep the kit in dry environment.
- Keep the kit away from direct sunlight.
- Do not freeze the device.
- If stored properly as mentioned above, the test device is stable through expire date printed on the sealed pouch.
- Keep out of the reach of children.
- When the pouch is opened, use the test within 60 minutes. Otherwise, the results may not be reliable.

PERFORMANCE CHARACTERISTICS

How sensitive is the test?

Negative results are expected in healthy non-pregnant women and healthy men. Healthy pregnant women have hCG present in their urine samples. The amount of hCG will vary greatly with gestational age (months of pregnancy) and between individuals. MidStream Pregnancy Rapid Test has a sensitivity of 25 mIU/mL. This correlates with WHO 3rd International Standards and is the amount of hCG present in urine of pregnant women as early as 1 day after the first day of missed period. To determine this value, 36 female urine samples tested negative for hCG other procedures (ELISA) were collected. Different amount of hCG were added to these negative urines (0,10, 25, 50 ,250, 500 mIU/ml). The results are as follow:

	0 mIU/ml	10 mIU/ml	25 mIU/ml	50 mIU/ml	250 mIU/ml	500 mIU/ml
Total	36	36	36	36	36	36
Negative	36	36	0	0	0	0
Positive	0	0	36	36	36	36
Invalid	0	0	0	0	0	0

Cross Reactivity Studies

These studies refer to studies performed to ensure that the MidStream Pregnancy Rapid Test does not give false positive results with homologous hormones (substances, structurally close to hCG). For this purpose, homologous hormones LH, FSH and TSH was added to negative urine. (LH: 300 mIU/ml, FSH: 1000 mIU/ml and TSH: 1000 mIU/ml. To

these samples, hCG was also added at different concentrations of 0, 20, 100 and 500 mIU/ml. These were tested with 3 lots of MidStream Pregnancy Rapid Tests. No cross reactivity was observed in the study.

Drug interaction studies

MidStream Pregnancy Rapid Test was checked for possible interference with substances that may appear in urine. These includes albumin, bilirubin and hemoglobin as well as common drugs. These substances were added into urine samples with different concentration of hCG. Results showed no significant interference was observed with following substances:

- Hemoglobin 10 mg/ml
- Albumin: 1 mg/ml
- Bilirubin: 100 mg/ml.
- Acetaminophen 20 mg/dl
- Acetyl salicylic Acid 20 mg/dl
- Ascorbic Acid 20 mg/dl
- Atropine 20 mg/dl
- Caefine 20 mg/dl
- Gentic Acid 20 mg/dl
- Glucose 2.0 mg/ml

Sensitivity and specificity (Clinical Trials)

To establish the sensitivity and specificity of MidStream Pregnancy Rapid Test relative to other qualitative urine hCG tests, a commercially available qualitative test kits was used to compare with MidStream Pregnancy Rapid Test for relative sensitivity and specificity in 312 urine samples. The samples were collected from 3 centers. The results are shown in following table..

MidStream Pregnancy Rapid Test	Atlas Link					PPV	NPV
	Positive	Negative	Total	% Agreement			
Positive	120	4	124	96.7			
Negative	0	188	188	>99			
Total	120	192	312				
% Agreement	99	97.9	60				
	Sensitivity	Specificity					

Accordingly, the sensitivity of the test is 99% and specificity is 97.9%.

References

- TIETZ *Textbook of Clinical Chemistry AND Molecular Diagnostics, 4th edition, 2006*
- Gnoth C, Johnson S. Strips of hope: accuracy of home pregnancy tests and new developments. *Geburtshilfe Frauenheilkd. 2014; 74(7):661-9.*
- Larsen J, Buchanan P, Johnson S, Godbert S, Zinaman M. Human chorionic gonadotropin as a measure of pregnancy duration. *Int J Gynaecol Obstet. 2013; 123(3):189-95.*
- Guideline to hCG Levels During Pregnancy, American pregnancy Association. **22 August 2017**
- Wilcox, A.J. et al, N. Eng. J. Med, Vol 319, 189, 1988

Symbols Description:

	In vitro diagnostic medical device		Manufacturer		Consult instructions for use
	Expiration date		Lot number		Disposable device, do not re-use
	Temperature limit (6°-30° C)		Keep away from sunlight		Keep in a cool, dry place
	Don't use if package is damaged		Contains sufficient for "11" tests		Discard in the dust bin after use.

CE 2854 Complies with the relevant EU legislation.

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1. Tietz *Textbook of Clinical Chemistry AND Molecular Diagnostics, 4. vydanie, 2006*
2. Gnath C, Johnson S. Strips of hope: accuracy of home pregnancy tests and new developments. *Geburtshilfe Frauenheilkd. 2014; 74(7):661-9.*
3. Larsen J, Buchanan P, Johnson S, Godbert S, Zinaman M. Human chorionic gonadotropin as a measure of pregnancy duration. *Int J Gynaecol Obstet. 2013; 123(3):189-95.*
4. Guideline to hCG Levels During Pregnancy, American Pregnancy Association. 22. august 2017
5. Wilcox, A.J. et al, N. Eng. J. Med, Vol 319, 189, 1988

Vysvetlenie symbolov :

	In vitro diagnostic medical device		Manufacturer		Consult instructions for use
	Expiration date		Lot number		Disposable device, do not re-use
	Temperature limit (5°-30° C)		Keep away from sunlight		Keep in a cool, dry place
	Don't use if package is damaged		Contains sufficient for "11" tests		Discard in the dust bin after use.

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